

VersaHaul VH-MS Display Stand Instructions

BNK, Inc. now offers an attractive display stand to showcase a VersaHaul mobility carrier in your showroom.

Features:

- Assembles in seconds; requires only two 3/8" bolts.
- One stand can display the VH-MS or VH-MS XL carriers.
- Hitch pin included.
- Will safely support a mobility scooter.

Draw attention to your prized mobility scooter by displaying it on a VersaHaul carrier and stand.

How to Use This Rack

For the **VH-MS Mobility Scooter Carrier** model, bolt the arms of the rack to the L-shaped frame using the two holes near the middle of the tube frame. The arms of the rack should be in a swept forward position. Set the VH-MS model on the display stand and use a hitch pin to secure (**see picture below**).

For the **VH-MS XL Mobility Scooter XL Carrier** model, bolt the arms of the rack to the L-shaped frame using the 3rd and 4th holes nearest to the end of the tube frame. The arms of the rack should be in a swept forward position. Set the VH-MS XL model on the display stand and use a hitch pin to secure (**see picture below**).



VH-MS Arrangement



VH-MS XL Arrangement